



TRUST VISOCU WITH YOUR EYES OPEN

visOcuTM

Sodium Hyaluronate

Hyaluronic Acid solution for intraocular use containing cannula for injection



www.phibio.de

visOcu

What is visOcu™



Dosage Info

- %1.0,%1.4,%1.8 Dispersive
- %1.8 Dispersive Cohesive
- %3.0 Cohesive



visOcu™ is a sterile, cohesive, isotonic, pyrogen-free, and viscoelastic preparation of highly purified sodium hyaluronate, dissolved in a buffered physiological solution (pH 6.7 to 7.5) for intraocular application. It is clear solution in a ready-to-use injection for single use. It is packaged in a sterile blister or edge-sealed pouch.

What is Hyaluronic acid?

Sodium hyaluronate is a biopolymer obtained from bacterial biofermentation that belongs to the class of glycosaminoglycans. It is a high molecular weight, is highly purified and possesses favorable physical and rheological properties for ophthalmic surgery. Hyaluronic acid is a biological and biocompatible substance. The viscoelastic properties of **visOcu™** allow lubrication, support and protection of ocular tissues during ophthalmic applications. It maintains the depth of the anterior chamber.

How to use VisOcu™?

If **visOcu™** was stored in the refrigerator, the syringe should be removed at least 2-3 minutes prior to use from the refrigerator. When the air has escaped from the ophthalmological Luer-Lock cannula, VisOcu™ can be carefully injected into the anterior chamber using the enclosed sterile single-use cannula or a comparable device. cannula. Dosage depends on the kind of surgery. To replace loss of viscoelastic by outflow or irrigation, several injections can be made. Remove **visOcu™** completely at the end of surgery using a suitable irrigation/aspiration

Sodium Hyaluronate



Characteristics and chemical compounds of visOcu™



Specifications	visOcu™ %1.0	visOcu™ %1.4	visOcu™ %1.6	visOcu™ %1.8	visOcu™ %3.0
Molecular weight* (MDa)	1.2-2.0	1.2-2.0	1.2-2.0	1.2-2.0	1.2-2.0
Viscosity* (mPas)	approx. 15.000	approx. 25.000	approx. 30.000	approx. 80.000	approx. 250.000
Osmolality	270-400	270-400	270-400	270-400	270-400
Storage	2°-25°	2°-25°	2°-25°	2°-25°	2°-25°
pH	6.7-7.5	6.7-7.5	6.7-7.5	6.7-7.5	6.7-7.5
Volume	1.0	1.0	1.0	1.0	1.0
Shelf life	24 months	24 months	24 months	24 months	24 months

Composition	visOcu™ %1.0	visOcu™ %1.4	visOcu™ %1.6	visOcu™ %1.8	visOcu™ %3.0
Sodium hyaluronate	10.00mg	14.00 mg	16.00mg	18.00mg	30.00mg
Sodium chloride	8.50mg	8.50mg	8.50mg	8.50mg	8.50 mg
Disodium hydrogen phosphate 2H ₂ O	0.563mg	0.563mg	0.563mg	0.563mg	0.563mg
Sodium dihydrogen phosphate 2H ₂ O	0.045mg	0.045 mg	0.045mg	0.045mg	0.045mg
Water for injection	q.s.	q.s.	q.s.	q.s.	q.s.

Hyaluronic Acid solution for intraocular use containing cannula for injection

TRUST VISOCU WITH YOUR EYES OPEN



Phibio GmbH
Klingerstr. 24, 60313
Frankfurt am Main, Germany



Phone: +4915207659955
Landline: +49(0)6940767397
e-mail: info@phibio.de

Ventus Sağlık Medikal San. Tic. LTD. ŞTİ.
Gazibey Mah.
Kızılay Sk. No:4/A
Malkara / Tekirdağ



Mustafa AKYÜZ : +90 530 721 51 97
Erhan ÇALIŞKAN : +90 532 666 67 02
info@ventusmedikal.com
www.ventusmedikal.com

w w w . p h i b i o . d e